Tray-Adhesive

for A-Silicones

DMG Chemisch-Pharmazeutische Fabrik GmbH
Elbgaustraße 248 · 22547 Hamburg · Germany
www.dmg-dental.com
092735/#1_2014-12

EN Instructions for use

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Instructions for use

English

Product description

Tray adhesive for adhesion between A-silicones with metal and plastic trays.

Indications

For adhesion addition curing silicone impression materials (PVS) with minimal or non-retentive impression trays (metal, plastic).

Contraindications

- ▶ Do not use the material in cases of allergies to any of the ingredients or, as can occur in rare cases, in the event of contact allergies.

Basic safety instructions

- For dental use only!
- Keep out of reach of children!
- Do not swallow. In the event of unintentional ingestion, seek advice from a doctor.
- Highly flammable. Keep away from naked flames.
- Do not inhale vapors. In the event of unintentional inhalation, likewise seek advice from a doctor.
- Avoid contact with skin and eyes! In the event of accidental contact, immediately rinse thoroughly with plenty of water and consult a physician if necessary.

Side effects

There have been no systemic side effects to date.

Instructions for use

- ▶ With plastic impression trays test the compatibility of the tray adhesive.
- ▶ Ensure that the impression tray is clean, free of grease and dry.

Recommended use

1. Using a brush, apply a thin, even layer of the tray adhesive to the inside of the impression tray.
2. Leave the tray adhesive to work for 3:00 min.
3. Add the impression material to the tray within 30 min.

Storage and shelf life

- ▶ Store in a dry place at room temperature (15 to 25 °C/59 to 77 °F)!
- ▶ Close bottle immediately after use.
- ▶ Do not use after the expiration date!

Disposal

- ▶ Dispose of in accordance with the regulations in the respective country.

Composition

Silicone resin, solvent (volatile paraffin)

Caution: U.S. Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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