## CLEARSHIELD 5% SODIUM FLUORIDE VARNISH

### INSTRUCTIONS FOR USE

**DESCRIPTION:** Kolorz ClearShield is a 5% sodium fluoride varnish which has a strong desensitizing action when applied to dental surfaces. It is used for the treatment of hypersensitive teeth and as a liner for cavity preparations. Because it is tolerant of moisture and saliva, it will adhere to moist surfaces and leaves a film of varnish that sets in the mouth and may allow visual control and verification.

**INDICATIONS/RECOMMENDED USE:** Use as a desensitizing agent to treat dentinal hypersensitivity and as a varnish to line cavity preparations underneath amalgam restorations.

**RECOMMENDED DOSE:** *Adults and children ages 12 years and up:* 0.4 -1.6mls 1 to 4 times per year. *Children under 12:* 0.4 - 0.8mls 1 to 4 times per year.

**DURATION OF USE:** Instruct the patient to allow **ClearShield** to remain in contact with tooth surfaces for a minimum of 4 hours. Also, instruct the patient to avoid hard foods, brushing or flossing teeth, or any beverage or oral rinses containing alcohol during the treatment period.

### INSTRUCTIONS TO APPLY CLEARSHIELD:

1. Peel open the foil lidding material
2. Remove brush and mix varnish until it reaches an even consistency
3. For best results, dry teeth prior to application by rubbing a 2x2 (or other disposable cotton sponge) along the tooth surfaces to remove any existing biofilm
4. Apply a very thin coat of varnish evenly to all of the desired tooth surfaces
5. Let the varnish dry for approximately 10 seconds and then instruct the patient to close their mouth
6. Varnish hardens on contact with saliva so the patient may leave immediately after application
7. When dry, any excess varnish may appear "spotty" and may slide off the tooth surface. This will not affect the efficacy of the product as there is still a full layer of varnish on the tooth
8. It is recommended that the patient be instructed to only eat soft foods and drink cold liquids for two hours after treatment

**PROPERTIES:** 1ml of **ClearShield** contains 50mg of sodium fluoride, which is equivalent to 22.6mg of fluoride in an alcohol-based suspension of natural resins. Sweetened with Sucralose. Does not contain Aspartame or Saccharin. Gluten free.

**PRECAUTIONS, WARNINGS AND RISKS:** For use by dental professionals only. For use in the oral cavity. Avoid direct ingestion.

**CONTRAINDICATIONS:** Avoid use of this product for patients with Ulcerative Gingivitis and Stomatitis.

**INTERACTIONS:** When **ClearShield** is applied, other fluoride preparations such as fluoride gels, should not be administered for 24 hours after application. The routine use of fluoride tablets or supplements should be interrupted for 2-3 days after treatment.

**KNOWN ADVERSE REACTIONS:** Dyspnea (labored breathing), although rare, has occurred in asthmatic patients. Edematous swellings have been reported in rare instances, particularly after application to extensive surfaces. Nausea may occur in patients with sensitive digestive systems, especially following prolonged treatment. If required, varnish film can be removed with a thorough brushing.

**MEDICINAL INGREDIENTS (per dose unit):** Each individual package contains 0.40 mL **ClearShield 5% Sodium Fluoride Varnish**.

**NON-MEDICINAL INGREDIENTS:** Rosin, beeswax-white, sucralose, xylitol, FD&C Red No. 40, melon flavor, bubblegum flavor, caramel flavor, spearmint oil, denatured alcohol.

**Distributed by:** **DMG America, LLC**  
242 S. Dean St., Englewood NJ 07631  
Tel: 800-662-6383

**Canadian Importer:** **Progress Therapeutics Inc.**  
320 Harry Walker Pkwy North, Suite #14  
Newmarket ON L3Y 7B4  
Canada

**License Holder:** **DMG America, LLC**  
242 S. Dean St., Englewood NJ 07631  
Tél: 800 662-6383

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**Fabriqué pour :** **DMG America, LLC**
**Titulaire de licence :** **DMG America, LLC**
**Importateur canadien :** **Progress Therapeutics Inc.**