



Kolorz Sixty Second Fluoride Gel

APF Topical Fluoride

Acidulated Phosphate Fluoride

1.23% Fluoride Ion

Available in: Triple Mint, Cherry Cheesecake, Cotton Candy, Blue Raspberry, Piña Colada

Indications:

For the topical application of fluoride to aid in the protection against dental caries.

Dosage & Administration:

Fill fluoride tray(s) 1/3 full with gel. Dry tooth surface and insert tray(s) in mouth. Have patient bite down for 60 seconds (or up to 4 minutes). Slight chewing motion provides interproximal coverage. Remove tray(s) and have patient expectorate excess gel. Advise patient not to eat, drink or rinse for 30 minutes.

Contains:

Sodium Fluoride in a proprietary acidulated phosphate flavored gel base. 1.23% Fluoride Ion from 2.72% Sodium Fluoride. Sweetened with Sucralose. Does not contain Aspartame or Saccharin. Gluten free.

WARNING:

Keep out of reach of children. Do not swallow. If dosage or less than 100mg is ingested, contact poison control. Store between 68-77° F. Keep from freezing. Federal (USA) law prohibits dispensing without prescription. For professional use only. Shake well before each use.

EMERGENCY RESPONSE:

Call Infotrac 1-800-535-5053

Distributed by DMG America LLC
242 South Dean Street
Englewood, NJ 07631
800-662-6383 • www.dmg-america.com