

INSTRUCTIONS FOR USE

LuxaGlaze



LuxaGlaze is a light-cure one-bottle varnish, based on multifunctional methacrylates for glazing the surfaces of provisional crowns, bridges and custom trays. LuxaGlaze's special properties are its complete setting reaction which leads to no surface inhibition layer after polymerization and its high gloss, hard abrasion resistant surface. Accumulation of debris and plaque are minimized when utilizing LuxaGlaze, which benefits the patients. It also improves resistance against staining. The glazed surface exhibits an exceptionally life-like appearance, enhancing esthetics.

FIELD OF USE

- Glazing of surfaces instead of high gloss polishing
- Enhancing esthetics of provisional crowns and bridges
- Improving stain resistance of the temporary
- Glazing of custom trays

APPLICATION

1. Clean or grind the desired area if not freshly polymerized.
 2. The varnish is applied to the areas to be treated in a thin, even layer with a soft brush. Avoid inclusion of air bubbles.
 3. LuxaGlaze is polymerized for 10 seconds with a suitable light-curing unit.
- Ensure that the areas are positioned as to allow for irradiation of areas that were not accessible for light curing previously.

LIGHT-CURING

Dental halogen light unit: Glaze and cure unit by unit for 10 seconds.

COMPOSITION

Silicon dioxide in a matrix of Bis-GMA and other dental resins, catalyst, stabilizer, pigments.

STORAGE

Store at $\leq 25^{\circ}\text{C}/77^{\circ}\text{F}$. Do not use after expiry date.
Close bottle immediately after use, to avoid evaporation and exposure to light.

PRECAUTIONS

- Do not use LuxaGlaze to adjust viscosity of composite restorative material.
- Do not use greasy or oily cleaning solutions.
- Dry with oil-free air.
- Material is light sensitive. Keep bottle away from bright light during use.
- Do not store materials in proximity of eugenol-containing products.

PACKAGING

1 bottle @ 5 ml liquid, 25 brushes

REF #####

Caution: Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

Exclusively Distributed in the United States and Canada by:
Zenith Brands Division of Foremost Dental Mfg. Co., Inc.
Telephone number: 1-800-662-6383

DMG Chemisch-Pharmazeutische Fabrik GmbH
Elbgaustraße 248 • 22547 Hamburg • Germany
Tel: +49 40-84 00 60 • Fax: +49 40-84 00 62 22
info@dmg-hamburg.de • www.dmg-hamburg.de



Keep away from children! For dental use only!

#####/05.01