

# Vitique

## Esthetic Cementation System







CE 0482

090399/10.2006

### I. Vitique-System: Use of Base syringe

#### Use of base syringe: light cure of Vitique

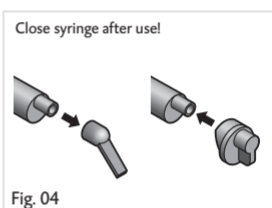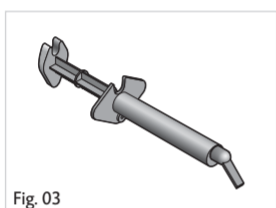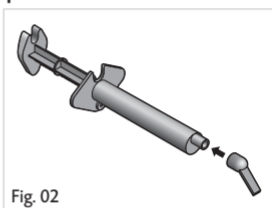
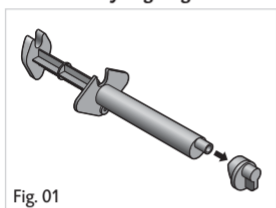


Please refer to detailed description in the following instructions for use.




### II. Vitique-system: Assembly of catalyst syringe

#### Use of base + catalyst syringe: dual cure of Vitique



##### Removal of syringe caps

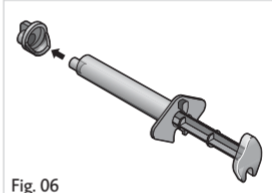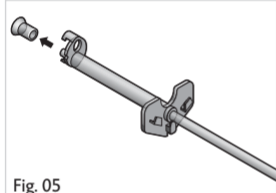

##### Assembly of base + catalyst syringe

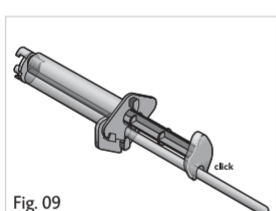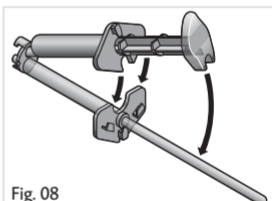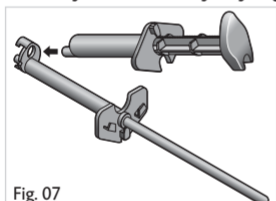


##### Attaching of Smartmix-tip

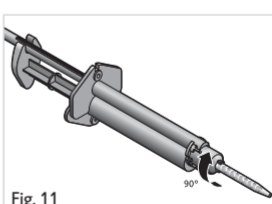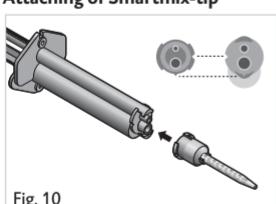


#### Please note!

It is essential to check the fill level of the catalyst syringe. For this, you can use the ratcheting system on the piston rod. When the ratcheting system is not visible, the catalyst syringe is completely empty.

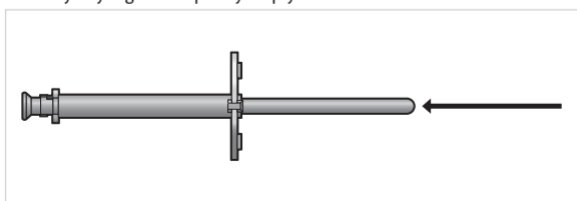

When the ratcheting system is not visible, the catalyst syringe is completely empty.

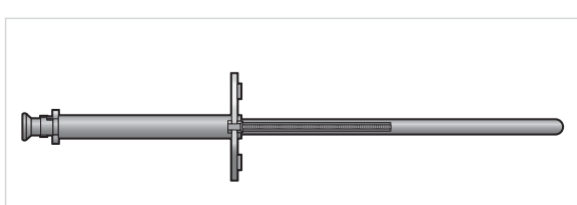

The ratcheting system is visible. The catalyst syringe contains material.

Please refer to detailed description in the following instructions for use.!





## Packaging

### Intro Kit

8 x 1 syringe @ 6 g paste Vitique base (A1\*, A2.5\*, A4\*, B1\*, Bleach Light, Transparent, White, Pink); 1 syringe @ 2.1 g paste Vitique catalyst, High Viscosity  
1 syringe @ 1.9 g paste Vitique catalyst, Low Viscosity; 8 x 1 syringe @ 3.9 g paste Vitique-Try-in (A1\*, A2.5\*, A4\*, B1\*, Bleach Light, Transparent, White, Pink);  
2 syringes @ 1.3 g paste Vitique-Try-in, Catalyst Shade; 20 Smartmix-Tips,  
100 Veneer-Tips REF 213013

### Refills

#### Vitique base

1 syringe @ 6 g Paste, 10 Veneer-Tips

A1*	REF 213038	A2.5*	REF 213039
A4*	REF 213040	B1*	REF 213041
Bleach Light	REF 213042	Transparent	REF 213043
White	REF 213044	Pink	REF 213045

#### Vitique catalyst

1 syringe @ 2.1 g paste, High Viscosity; 10 Smartmix-Tips REF 213046

1 syringe @ 1.9 g Paste, Low Viscosity; 10 Smartmix-Tips REF 213047

#### Vitique-Try-in

1 syringe @ 3.9 g paste, 10 Veneer-Tips

A1*	REF 213048	A2.5*	REF 213049
A4*	REF 213050	B1*	REF 213051
Bleach Light	REF 213052	Transparent	REF 213053
White	REF 213054	Pink	REF 213055

1 syringe @ 1.3 g paste, 10 Smartmix-Tips  
Catalyst Shade REF 213056

#### Accessories

100 Veneer-Tips REF 213057

50 Smartmix-Tips Brown 4:1 REF 213058

\* Shades according to Vita - Vita is a registered trademark of Vita Zahnfabrik H. Rauter GmbH

Federal Law (USA) restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

Keep away from children! For dental use only!

