

## **INSTRUCTIONS FOR USE**

---

### **ZEKRYA GINGIVAL PROTECTOR**

#### **THE ZEKRYA GINGIVAL PROTECTOR allows:**

- retraction of the gum without injury-risk,
- complete protection of the gum during tooth preparation,
- good visibility.

#### **TECHNICAL FEATURES:**

- gingival protectors are sterilizable and replaceable.
- swivel joint allows an easy adjustment to any desired position.
- the flexible arc of the protectors adapts to fit all tooth necks.

#### **TIPS FOR USE:**

- place the protector on the mandrel in the desired position and fix it firmly,
- protect the patient's lips by placing a cotton roll between gum and lips,
- press the protector firmly against the tooth neck so that it fits,
- use the spray in order to prevent overheating.

#### **Order-numbers:**

REF G 0260	Set with	10 protectors N° 1 10 protectors N° 2 1 swivel mandrel
REF G 260A	Swivel mandrel alone	
REF G 260B	Gingival protectors (10 pieces) N° 1 or 2	