DMG

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Instructions for use

English

TempoCem and TempoCem Soft are automix cements based on zink oxyde/eugenol. They are used as cements for temporary luting of all kinds of temporaries, e.g. temporary crowns, bridges, inlays and onlays.

TempoCem flows easily and thus allows an unobjectionable placing of the temporary. It is resistant to saliva and stable in the mouth. It

withstands the stress of mastication sufficiently, but allows an easy removal of the temporary. Due to an added modifier, TempoCem Soft is adjusted to a slower pace and has a decreased final hardness. This allows easier removal of temporary crowns and bridges with very high friction.

Indication

Temporary luting of provisional and permanent crowns and bridges, inlays and onlays.

Timing

TempoCem

Working time

at room temperature approx. 1:00 minute

Setting time in the mouth 4 minutes

TempoCemSoft

Working time

at room temperature approx. 2:30 minutes


Setting time in the mouth 8 minutes

Application system

TempoCem-Automix: see Automix System

TempoCem-Smartmix: see Smartmix System

Automix System

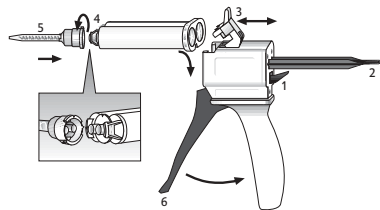1. Push up the lever (1) at the back of the dispensing gun and pull back the slide (2) completely. Lift the plastic lever (3) on the top of the gun, insert cartridge and lock in by pushing down the lever (3). Ensure that the notch on the cartridge is aligned with the notch on the dispensing gun.


2. Remove the cartridge cap (4) by turning 90° anti-clockwise and discard it. Attach a new mixing cannula (5). Note the corresponding notches. Turn the cannula 90° clockwise to lock it.
3. The material is mixed in the mixing cannula by pulling the trigger (6) of the dispenser and can be applied directly.

When using a cartridge for the first time, express a pea-sized amount of material and discard it.


Leave the used cannula on the cartridge to serve as a cap until the next use.

4. Push up the lever (1) at the back of the dispensing gun and pull the slide (2) back completely. Lift the plastic lever (3) and remove the cartridge.

Smartmix-System

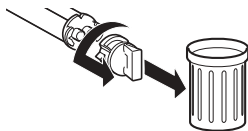
1. Remove the cap (4) by turning 90° anti-clockwise and discard it.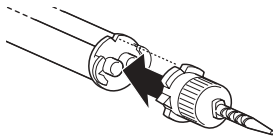

2. Attach a new Smartmix Tip. Ensure that the notches on the syringe and on the tip are aligned.

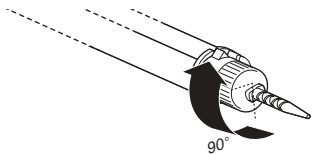3. Lock the tip by turning it 90° clockwise.
4. The material is mixed in the cannula while being dispensed and can be applied directly.

Leave the used cannula on the cartridge to serve as a cap until the next use.

Recommended Use

1. Apply TempoCem into the dry temporary bridge or crown.
2. Affix the temporary with gentle pressure to the dry prepared tooth.
3. Excess cement can be removed easily with a probe in large chunks. Remove interdental excess by flossing carefully. The light color of

the material makes it easy to distinguish from teeth.

Important Notes

- Cements containing eugenol can inhibit the polymerization of composites, resin-based luting cements and bonds. Therefore, please check if the use of the eugenol-free, temporary cement TempoCemNE is more advantageous.
- Certain studies assume a correlation between eugenol and pulp damages. Therefore, check the necessity of a pulp protection in each individual case.

Side Effects

There have been no known systemic side effects to date. However, you should not use the material where allergies to one of the components are known to exist or, in the rare cases, where contact allergies exist.

Composition

Catalyst paste: natural resins, eugenol, additives

Base paste: zinc oxide, paraffin, additives

Storage

Store only at room temperature (18-25 °C/
64 to 77 °F).

Do not use after the expiration date.

Classification

ISO 3107:1988 (because of the automatic mixing, the curing time is shorter than usual).

Packaging

Refill packs

1 cartridge @ 63 g paste, 35 Automix-Tips

TempoCem REF 110247*

TempoCem-Soft REF 110265*

2 Smartmix double syringes containing 11 g
paste, 20 Smartmix-Tips

TempoCem REF 212129*

*Not available in the United States

Accessories

1 Automix-Dispenser Type 25 1:1 REF 110253

50 Automix-Tips Blue 1:1 REF 110266

50 Smartmix-Tips Short REF 212040

Only available in the United States

Introductory Packs

1 cartridge @ 63 g paste, 50 Automix-Tips

1 Automix-Dispenser REF 110246

Refills

1 cartridge @ 63 g paste, 40 Automix-Tips

TempoCem REF 110300

TempoCem-Soft REF 110303

2 Smartmix double syringes containing 11 g
paste, 10 Smartmix-Tips

TempoCem REF 212129

Caution: Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

Keep away from children! For dental use only.

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