

# StatusBlue™

## The Automix Alginate Replacement

■ The faster, easier alternative to alginates



### Advantages

- Firm consistency for slump-free tray dispensing.
- Sets quickly for greater patient comfort.
- Stays dimensionally stable.



 **DMG**  
AMERICA

# StatusBlue puts an end to the mess, trouble and lost chair time caused by alginates.

## Time To Take Impression\* (in minutes)

	Prep time	Time to set in mouth	TOTAL TIME
Alginates	1:45	4:45	6:30
StatusBlue	0:45	1:45	2:30

Time saved with StatusBlue = 4 mins. per impression


### DO THE MATH:

If you take an average of 4 impressions a day, the time you save with StatusBlue earns you up to **55 extra hours of valuable chair time each year!**

\* Time based on manufacturers' internal testing.



**PUSH**

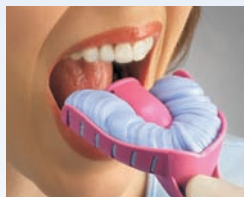

**PLACE**



**POUR**

## There is simply no faster, easier choice for situation impressions.

Top lecturers teach that alginates – with their time-consuming hand-mixing, mess and cleanup – are strictly old school. The ideal choice to create situation impressions for temporaries, model-cast dentures, orthodontic models and opposing jaw models is StatusBlue.

## Firm consistency for slump-free tray dispensing.

StatusBlue has an innovative A-silicone formula for a firm consistency and better patient acceptance than alginates. Fast, easy dispensing options include 50ml cartridges for Automix guns and 380ml cartridges for MixStar.

## Sets quickly for greater patient comfort.

StatusBlue sets in the mouth in just 1 minute 45 seconds, for better patient comfort. And its unique viscosity allows it to flow better under bite pressure, so it captures exceptional accuracy and detail.

## Stays dimensionally stable.

Unlike alginates, StatusBlue impressions stay dimensionally stable and do not have to be kept moist – so there's no pressure to "pour it or lose it." You can pour the model today, tomorrow or next week! You can even cold sterilize StatusBlue impressions and re-pour as needed.

## Physical Properties: StatusBlue

Composition	Vinyl polysiloxanes
Consistency	Type 2, Medium Body
Compression set	≤ 0.7%
Base/Catalyst Ratio	4:1
Total Working Time	1:15 minutes
Recommended Time in Mouth	1:45 minutes
Dimensional Change (after 24 hr)	≤ 0.4%
Shore A Hardness	ca. 40
Strain in Compression	≈ 7.0%

## Ordering Information:

999826	StatusBlue for Automix Guns Economy Tub (7 +1 FREE 50ml Cartridges, 16 Automix Tips)
999693	StatusBlue MixStar (1-380ml Cartridge and 10 MixStar Tips)
110411	Type 50 Automix Gun (10:1)
999797	StatusBlue Automix Gun Tips (Bag of 25)

For more information, call us at **1-800-662-6383**. To find out more about all of our innovative dental products, visit our website at [www.dmg-america.com](http://www.dmg-america.com)



## DMG America

242 South Dean Street  
Englewood, New Jersey 07631

P: 800-662-6383 • 201-894-5505

F: 201-894-0213 • [www.dmg-america.com](http://www.dmg-america.com)

