



## A Temporary Cement That Does Not Compromise Esthetics

Written by Albert Ambriz, DDS, MDT

Without compromising aesthetics, TempoCemID has made temporary cementation easier than ever before. Previously, using a clear temporary cement would leave a gray cast under provisionals. Now, with TempoCemID, provisionals are left with a more natural result. Its dual cure setting action makes for an easy clean up as all excess material comes off in one piece.

### Case

Patient previously treated for root canal treatment due to deep decay. The procedure included a prep, temporary and a final impression of tooth #4.

Using a triple tray impression technique, StatusBlue, an alginate replacement material, was used for the fabrication of the provisional. For the creation of the provisional, Luxatemp Ultra, was used.

The final impression (yellow/blue) was made using Honigum Pro Heavy and Honigum Pro Light body to achieve the greatest detail.



Temporary impression with StatusBlue (light blue). Final impression with Honigum Pro Heavy & Honigum Pro Light Body (yellow/blue)



Tooth #4 preparation for Zirconia restoration



Initial temporary cementation of Luxatemp Ultra provisional on tooth #4, using TempoCemID cement



Extruded TempoCemID upon fully seating provisional



Light cure excess TempoCemID cement for 2 seconds on buccal and lingual surface for initial set of excess cement



TempoCemID cement excess removal in one piece



Final results after temporarily cementing Luxatemp Ultra provisional with TempoCemID cement