



QUICK CASE: Carla Cohn, DMD



Icon® Caries Infiltrant Smooth Surface Treatment

OVERVIEW

- ↗ 7 year old female, teeth #11, #21
- ↗ Molar incisor hypomineralization



Before treatment



Immediately after treatment



18 months after treatment

PRE-TREAT

- 1**
 - ↗ Pumiced
 - ↗ Abraded with high speed FG diamond (only use when necessary)

ETCH

- 2**
 - ↗ Etched 3 times

INFILTRATE

- 3**
 - ↗ Allowed infiltrant to penetrate for 15 mins.
 - ↗ No light exposure prior to light cure

Be sure to follow all of the manufacturer's instructions for use. All steps were performed but not included in this case document.

VISIT [DMG-CONNECT.COM/ARTICLES](https://www.dmg-connect.com/articles)
FOR MORE CASES

