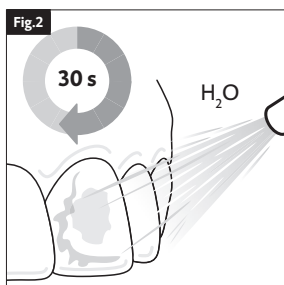
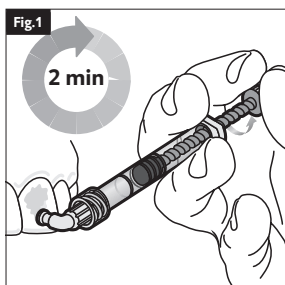
Icon Smooth Surface Etch

DMG Chemisch-Pharmazeutische Fabrik GmbH
Elbgaustraße 248 · 22547 Hamburg · Germany · www.dmg-dental.com
092918/2021-12

Handling

Instructions for use

English

Product description

The etching gel Icon Smooth Surface Etch forms part of infiltration treatment with Icon. For the entire infiltration treatment, the following instructions for use must be strictly observed:
Icon Smooth Surface

Indications for use

Removing the pseudointact surface layer in connection with infiltration treatment with Icon.

Limitations of use

Sufficient drying and protection of the gingiva not possible.

Contraindications

Do not use this material:
in cases of allergies to any of the ingredients or in cases of contact allergies.

Patient target group

People treated in the course of a dental procedure, e. g. patients with non-cavitated carious lesions, fluorosis or idiopathically or traumatically caused spots.

Intended users

Dentist

Notes for use

- If the material is applied into the patient's mouth with a single use applicator, the single use applicator must only be used on one patient for hygiene reasons.
- The etched surface must not be touched or contaminated with saliva or moisture until the treatment resumes. If contamination occurs after drying re-etch for approx. 10 seconds and repeat drying with Icon Dry.
- For effective treatment results a sufficiently dry working area is critical. Therefore, appropriate measures for complete isolation must be taken. Do not use rubber dams made from thermoplastic elastomers such as Flexi-Dam (Coltène Whaledent/Hygenic).

Recommended use (for image, see »Handling«)

Application of Icon Smooth Surface Etch:

- Screw the Vestibular Tip onto the Icon Smooth Surface Etch syringe.
- Apply an ample amount of Icon Smooth Surface Etch onto the lesion site by turning the syringe shaft carefully, let set for 2 minutes and activate it by moving (Fig.1). Remove excess material with dental wadding.
- Suck off Icon Smooth Surface Etch and rinse with water for at least 30 s. Then dry using oil-free and water-free air (Fig. 2).

Interactions

Do not use rubber dams made from thermoplastic elastomers such as Flexi-Dam (Coltène Whaledent/Hygenic).

Residual risks/side effects

- There are no known side effects to date. The residual risk of hypersensitivity in relation to allergies to components of the material cannot be ruled out.
- Contact of Icon Smooth Surface Etch with the oral mucosa causes a white surface coloration. This will subside after a few days.

Warnings / precautions

- Keep out of the reach of children.
- Caution! Icon Smooth Surface Etch can cause chemical burns
- Avoid contact with soft oral tissue, eyes, and skin. In case of inadvertent contact rinse with copious amounts of water, and, if necessary, consult a physician.
- Limit the application to the surface to be treated. Avoid contact of the etching gel with the gingiva, exposed dentine, root element and adjacent teeth.
- Use only the enclosed syringe and application tip to apply the etching gel in the patient's mouth.
- Dispose of application tip after use. Reattach syringe closure.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Performance characteristics of the device

pH value ≤ 0

Storage / disposal

- Store in a dry place at 2 to 25°C (36 to 77°F)!
- Protect against exposure to direct sunlight!
- Do not use after the expiration date!

Composition

Hydrochloric acid, pyrogenic silicic acid, surface-active substances

Note: A glossary of all symbols is displayed at: <http://www.dmg-america.com/literature>

Caution: U.S. Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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