# TempoCemNE

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English

### Product description

TempoCemNE is a eugenol-free automix cement based on zinc oxide that is indicated for temporary cementation crowns and bridges, inlays and onlays, as well as all types of temporary restorations.

### Indications

#### TempoCemNE

- Temporary cementation of provisional and permanent crowns and bridges, inlays and onlays.

### Contra-indications

Do not use the material in cases of allergies to any of the substances contained or, as can occur in rare cases, in the event of contact allergy.

### Basic safety instructions

- Only for dental use!
- Keep out of reach of children!
- Avoid contact with skin and eyes! In the event of accidental contact, rinse thoroughly with plenty of water immediately and consult a physician if necessary.
- Contains rosin

### Application System

- Automix cartridge: see "Using the Automix cartridge"
- Smartmix syringe: see "Using the Smartmix syringe"

**Please note:** When using a cartridge or syringe for the first time, squeeze out a pea-sized amount of material and discard it.

### Timing

#### TempoCemNE

≈1:00 min	Working time
4:00 - 5:00 min	Curing time in the mouth


**Please note:** The times given apply at a room temperature of 23 °C and a normal relative air humidity of 50 %. Higher temperatures shorten, lower temperatures increase these times.

### Recommended Application

#### Temporary cementation

1. Apply TempoCemNE to the dry temporary restoration.
2. Insert the temporary restoration within 1:00 min of start of mixing and attach to the dry, prepared tooth using slight pressure.
3. After curing, remove excess cement with a probe.
4. It is easy to remove excess cement in large chunks.
5. Remove interdental excess by flossing carefully.
6. The light color of the material makes it easy to distinguish from teeth.

### Using the Automix cartridge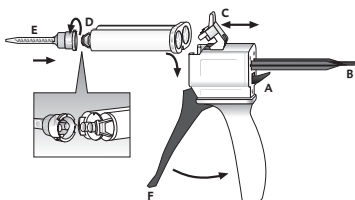

### Insert cartridge

1. Push up the lever [A] at the back of the Automix dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C], insert the cartridge and lock it using the plastic lever [C].

**Please note:** Make sure that the notches on the cartridge line up with those on the Automix dispenser.

### Attach mixing tip

1. Remove the cartridge cap [D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

**Please note:** Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.

### Apply material

- ▶ When using a cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- ▶ The material is mixed in the mixing tip by pulling the trigger [F] of the Automix dispenser and it is then applied directly.


**Please note:** After the application, leave the used mixing tip on the cartridge to cap unused material!

### Remove cartridge


1. Push up the lever [A] at the back of the Automix dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C] and remove the cartridge.

### Using the Smartmix syringe

---

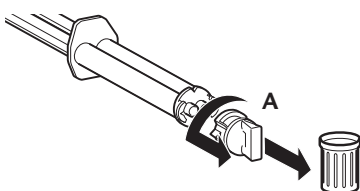
1. Before putting on the mixing cannula, remove the cap [A] or used mixing tip after twisting it 90° counterclockwise, and then discard it.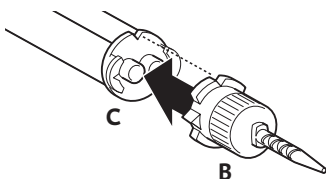

2. Attach a new mixing tip [B].

**Please note:** Make sure that the notches on the mixing tip line up with those on the Smartmix syringe [C].

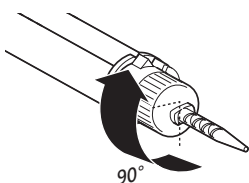3. Turn the mixing tip clockwise 90° to lock it.
4. The material is mixed in the cannula when it is expressed and it can be applied directly.

**Please note:** When using a syringe for the first time, squeeze out a pea-sized amount of material and discard it!

Leave the used mixing cannula on the Smartmix syringe after use. It serves as a cap!

### Disinfection of the Automix dispenser

---

The Automix dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute). The Automix dispenser is also autoclavable.

### Side effects

---

Isolated cases of hypersensitivity or contact allergy to components of the material cannot be excluded.

### Composition

---

- Catalyst paste: Natural resins, fatty acids, additives
- Base paste: Zinc oxide, paraffin, additives
- Available in 5 ml and 25 ml

### Classification

---

ISO 3107:2011, type 1 (because of the automatic mixing, the curing time is shorter than usual)

### Storage and shelf life

---

- ▶ Store at room temperature (15 to 25 °C/59 to 77 °F) in a dry place!
- ▶ Do not store in the refrigerator!

- ▶ Protect against exposure to direct sunlight!
- ▶ After the application, leave the used mixing tip on the Automix cartridge to cap unused material!
- ▶ Leave the used mixing cannula on the Smartmix syringe after use. It serves as a cap!
- ▶ Do not use after the expiration date!

**Note:** A glossary of all symbols displayed is at: <http://www.dmg-america.com/literature>

**Caution:** U.S. Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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