

INSTRUCTIONS FOR USE

VARIABLE SPEED AMALGAMATOR

AMALGAMATOR DIRECTIONS

The most accurate trituration guide for any specific alloy must be obtained from the manufacturer. Below find a guide that will help in the determination of an acceptable trituration time for your situation.

Powder or Pellets

Metal Pestle	6- 8 sec.	one tablet
	8-10 sec.	two tablets
	10-12 sec.	three tablets
Plastic Pestle	8-10 sec.	one tablet
	10-12 sec.	two tablets
	12-14 sec.	three tablets

Capsulated Alloy

Brands with Pestle	4- 6 sec.	single load
	6- 8 sec.	double load
	8-10 sec.	triple load
Brands without Pestle	6- 8 sec.	single load
	8-10 sec.	double load
	10-12 sec.	triple load

Variations between machines and variations in line voltage may affect proper time for your machine. These times are to be used as a guide and should be adapted to your situation.

CAUTION: Risk of injury is possible if the cover is not in the down position when unit is operating.