

# ZEKRYA GINGIVAL PROTECTOR



SWISS MADE

## THE ZEKRYA GINGIVAL PROTECTOR ALLOWS:

- Retraction of the gum without injury-risk
- Complete protection of the gum during tooth preparation
- Good visibility

## TECHNICAL FEATURES:

- Gingival protectors are sterilizable and replaceable
- Swivel joint allows an easy adjustment to any desired position
- The flexible arc of the protector adapts to fit all tooth necks
- Retracts soft tissue while protecting gingiva during rotary instrumentation.

## TIPS FOR USE:

- Place the protector on the mandrel in the desired position and fix it firmly
- Protect the patient's lips by placing a cotton roll between gum and lips
- Press the protector firmly against the tooth neck so that it fits
- Spray with water in order to prevent overheating while using a rotary instrument