



StatusBlue

The automix alginate replacement



End the mess, trouble and lost chair time caused by alginates

The fastest and easiest choice for situation impressions

Clinicians agree that alginates—with their time-consuming hand-mixing, mess and cleanup—are old school. The ideal choice to create situation impressions for temporaries, model-cast dentures, orthodontic models and opposing jaw models is StatusBlue.



Slump-free tray dispensing

Its innovative A-silicone formula offers a firm consistency and better patient acceptance over alginates. The material's unique viscosity also provides better flow under bite pressure to capture exceptional accuracy and detail.

Dimensional stability

Unlike alginates, StatusBlue stays dimensionally stable and doesn't have to be kept moist, so there's no pressure to "pour it or lose it." You can even cold sterilize it and re-pour as needed.

PHYSICAL PROPERTIES

Composition: Vinyl polysiloxanes	Compression set: ≤ 0.7%
Consistency: Type 2, Medium Body	Total Working Time: 1:15 minutes
Strain in Compression: ≈ 7.0%	Recommended Time in Mouth: 1:45 minutes
Shore A Hardness: ca. 40	Dimensional Change (after 24 hr): ≤ 0.4%
Base/Catalyst Ratio: 4:1	

TIME TO TAKE IMPRESSION* (IN MINS.)

	PREP TIME	SET TIME	TOTAL TIME
ALGINATES	1:45	4:45	6:30
STATUSBLUE	0:45	1:45	2:30

TIME SAVED WITH STATUSBLUE = 4 MINS. PER IMPRESSION

That's 55 extra hours of valuable chair time each year!

(*Time based on manufacturer's internal testing and avg. 4 impressions/day)



Initial situation.



Application of the DMG Tray Adhesive.



Homogeneous and bubble-free StatusBlue application into the border-lock tray.



Precise reproduction of the oral situation.



Filling the StatusBlue impression with Luxatemp from DMG.

Ordering Information

Item #	Product	Contents
999826	StatusBlue Automix Canister	7 +1 FREE 50ml Cartridges, 16 Automix Tips
999693	StatusBlue MixStar	1-380ml Cartridge, 10 MixStar Tips
110411	Type 50 Automix Gun (10:1)	1 Gun
999797	StatusBlue Automix Gun Tips	Bag of 25

DMG America, LLC
 65 Challenger Road, Suite 340, Ridgefield Park, NJ 07660
 tel: 800.662.6383 email: info@dmg-america.com
 web: www.dmg-america.com

