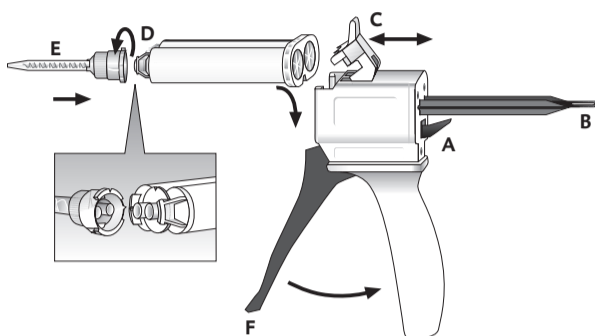
Automix Dispenser

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English

Product description

The Automix Dispenser is a mechanical tool for applying two-component dental materials. It enhances the applied manual force and transfers it to the plunger via a feed mechanism. The plunger transfers the feed to the pistons of the cartridge, which allows for an easy and precise application of the material from the cartridge.

Indications for use

Transfer and enhancement of the applied manual force from the hand to the pistons of a cartridge for an easy and precise application of two-component dental materials.

Limitations of use

To be used only for cartridges suitable for the respective Automix Dispenser.

Contraindication

Do not use the material where there are known allergies to any of the components or contact allergies.

Patient target group

Persons receiving treatment as part of a dental procedure. The indications for the use of the Automix Dispenser are derived from the materials to be used for dental treatment. Refer to the manufacturer's instructions for the use of the selected dental material for the indications for use.

Intended user

Dentist, dental technician, dental assistant staff

Notes for use

If the material is applied to the patient's mouth with a single use applicator, the single use applicator must be used for this one patient only for hygienic reasons.

Using the Automix Dispenser (for image, see "Handling")

1. Push up the lever [A] at the back of the Automix Dispenser and pull back the slide [B] all the way.
2. Lift up the plastic latch [C], insert the cartridge and lock it using the latch [C].

Note: Make sure that the notches on the cartridge line up with those on the Automix Dispenser.

3. Remove the cartridge cap [D] or used mixing tip after turning it 90° counterclockwise and discard it.
4. Attach a new mixing tip [E].

Note: In order to achieve an optimum mixing result, the manufacturer recommends the use of the mixing tips available from the manufacturer.

5. Twist the mixing tip clockwise by 90° to lock it.
6. Applying the material

Note: When using a cartridge for the first time, squeeze out a pea-sized amount of material and discard it.

7. Mix the material in the mixing tip by pressing the lever [F] on the Automix Dispenser and apply the material directly.

Note: After the application, leave the used mixing tip on the cartridge as a seal!

Cartridge removal

1. Push up the lever [A] at the back of the Automix Dispenser and pull back the slide [B] all the way.
2. Lift up the plastic latch [C] and remove the cartridge.

Disinfection of the Automix Dispenser

The Automix Dispenser can be reused after disinfection, although it should be replaced if there are visible signs of wear.

Disinfection can be carried out using a commercially available immersion disinfectant. DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute).

Sterilization

Use the validated sterilization parameters specified in the table below:

Procedure	Steam sterilization cycle with dynamic air removal
Region	USA
Exposure time	4 min
Temperature	132 °C (269.6 °F)
Drying time	20 min

Interactions

When using the Automix Dispenser, interactions may result from the material used for dental treatment. Refer to the manufacturer's instructions for the use of the selected dental material.

Residual risks/side effects

There are no known side effects to date. The residual risk of hypersensitivity to components of the material cannot be ruled out.

Warnings/precautions

- Keep out of the reach of children!
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer (productsafety@mtcompanies.com) and to the responsible registration authorities.

Product performance characteristics

Dispenser for dual cartridges

Storage/disposal

Store in a dry place at 2 to 25 °C (36 to 77 °F)!

Note: A glossary of all symbols is displayed at: <http://www.dmg-america.com/literature>

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