

# ZEKRYA GINGIVAL PROTECTOR



SWISS MADE

## THE ZEKRYA GINGIVAL PROTECTOR ALLOWS:

- Retraction of the gum without injury-risk
- Complete protection of the gum during tooth preparation
- Good visibility

## TECHNICAL FEATURES:

- Gingival protectors are sterilizable and replaceable
- Swivel joint allows an easy adjustment to any desired position
- The flexible arc of the protector adapts to fit all tooth necks
- Retracts soft tissue while protecting gingiva during rotary instrumentation

## TIPS FOR USE:

- Place the protector on the mandrel in the desired position and fix it firmly
- Protect the patient's lips by placing a cotton roll between gum and lips
- Press the protector firmly against the tooth neck so that it fits
- Spray with water in order to prevent overheating while using a rotary instrument

Product Number	Description
61096	Zekrya Kit: 10 Small Arcs 10 Large Arcs 1 Gingival Protector
61097	Bag of 10 Small Arcs
61098	Bag of 10 Large Arcs

 **ZenithDental™**  
Enhancing dentistry with education and technology®

Distributed by DMG America LLC  
242 South Dean St., Englewood NJ 07631  
(800) 662-6383

# ZEKRYA

## PROTECTEUR GINGIVAL



FABRIQUÉ EN SUISSE

### LE PROTECTEUR GINGIVAL ZEKRYA PERMET :

- La rétraction de la gencive sans risque de blessure,
- La protection complète de la gencive lors de la préparation dentaire,
- Une bonne visibilité.

### CARACTÉRISTIQUES TECHNIQUES :

- Les protecteurs gingivaux sont stérilisables et remplaçables.
- Swivel joint allows an easy adjustment to any desired position
- The flexible arc of the protector adapts to fit all tooth necks
- Retracts soft tissue while protecting gingiva during rotary instrumentation

### CONSEILS D'UTILISATION:

- Placez le protecteur sur le mandrin dans la position souhaitée et fixez-le fermement.
- Protégez les lèvres du patient en plaçant un rouleau de coton entre la gencive et les lèvres.
- Appuyez fermement le protecteur contre le col de la dent de manière à ce qu'il s'adapte.
- Spray with water in order to prevent overheating while using a rotary instrument

**Numéro du produit**  
61096

**Description**  
Kit Zekrya  
10 petits arcs  
10 grands arcs  
1 protecteur gingival  
Sachet de 10 petits arcs  
Sachet de 10 grands arcs

61097  
61098

 **ZenithDental™**  
Enhancing dentistry with education and technology®

Distribué par DMG America LLC  
242 South Dean St., Englewood NJ 07631  
(800) 662-6383