



# LuxaCore Z-Dual


Quick guide for Post cementation and core build-up

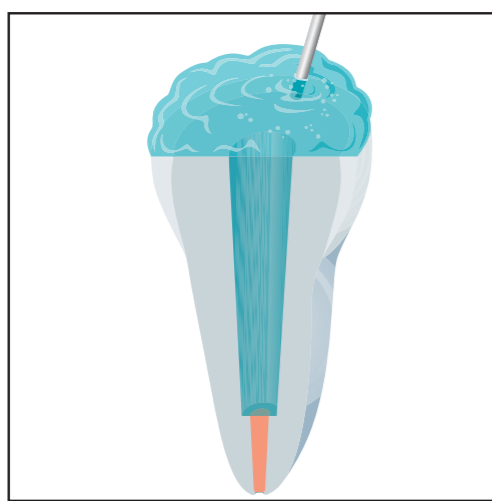


Etch enamel  
(20–60 s)  
and dentin  
(max. 15 s).

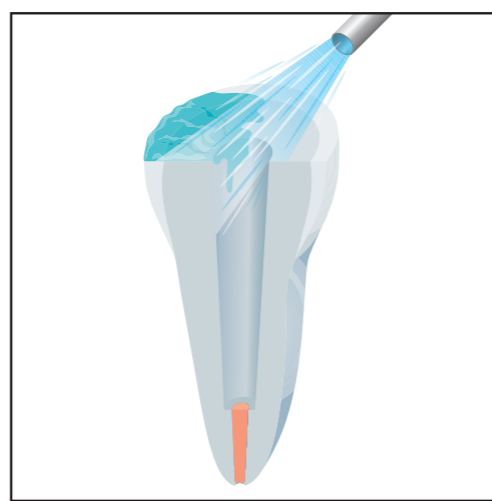


Rinse with water  
for at least 15 s.

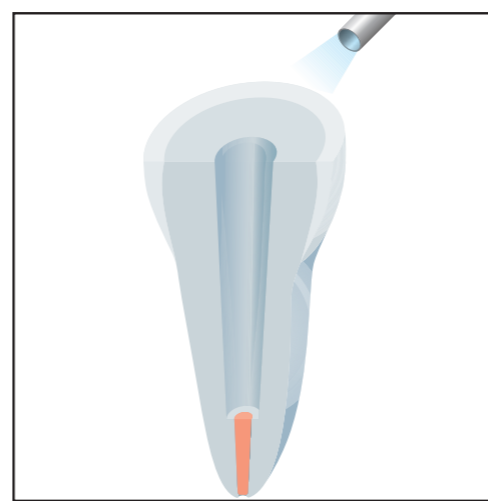


Remove excess  
water. Do not  
desiccate tooth  
substance.

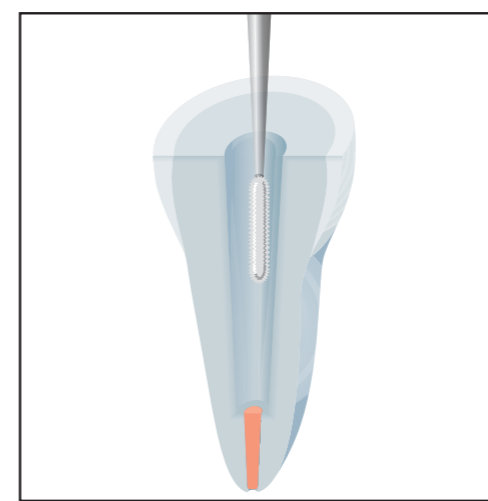


Apply  
LuxaBond-Total Etch  
into the coronal cavity  
and the root canal.

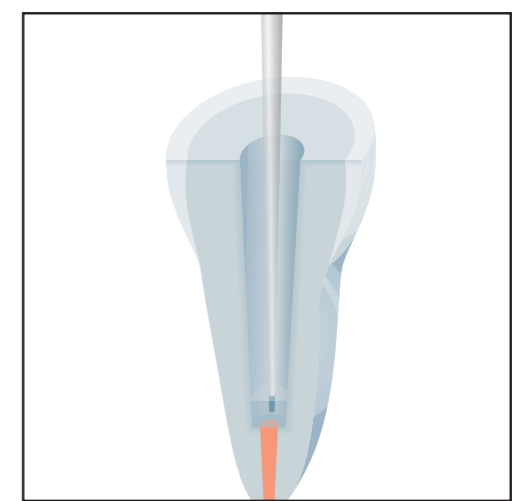


Remove excess  
material.  
Air blow gently.

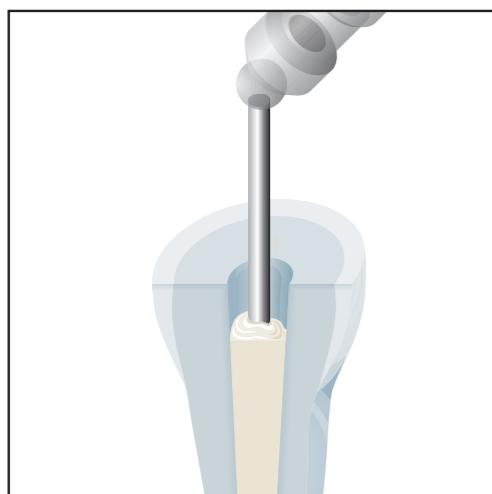


Apply  
LuxaCore Z-Dual  
into the canal lumen.

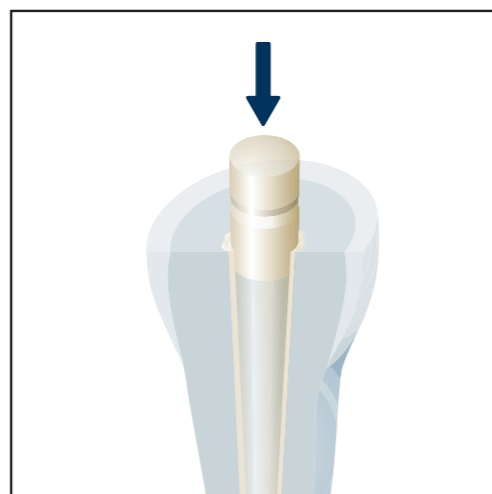


Insert the  
LuxaPost.

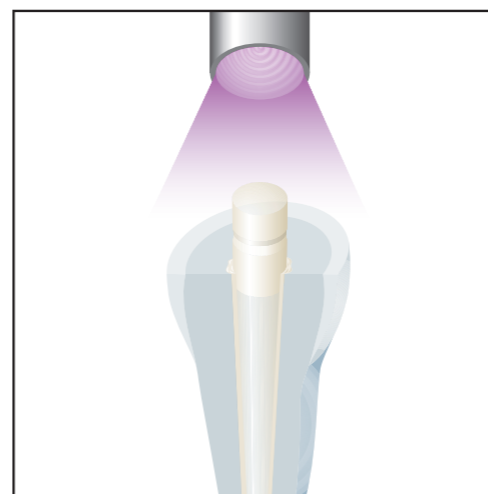

Optional  
light-curing: 20 s  
Self-curing: ≈ 5 min

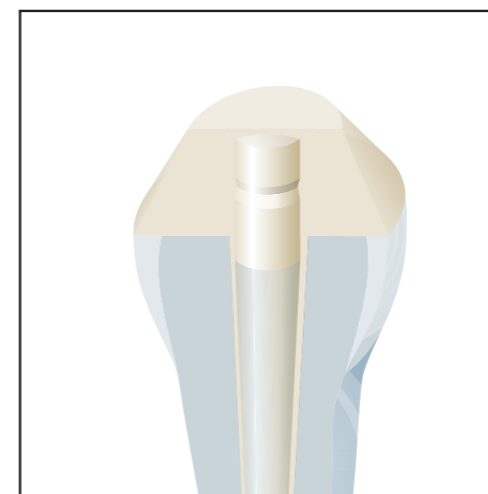

Build-up core with  
LuxaCore Z-Dual.

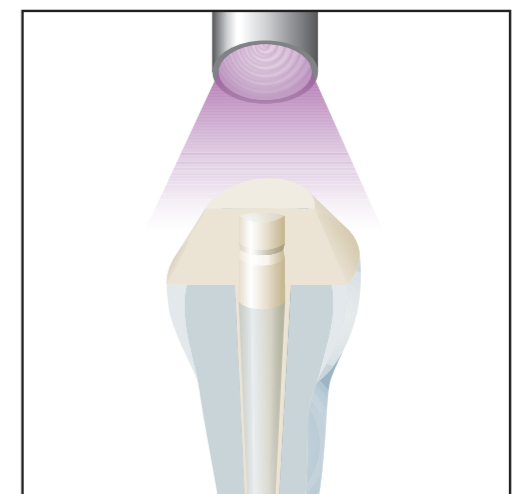

Optional light-curing:  
Layer ≤ 4 mm: 40 s  
Layer ≤ 2 mm: 20 s  
Self-curing: ≈ 5 min