

## LuxaPost

Quick guide for Use with LuxaBond-Total Etch and LuxaCore Z-Dual



Determine preparation depth and post length.



Prepare the root canal with the corresponding drill.



Clean and dry. Try the post in.



Etch enamel (20 - 60 s) and dentin (max. 15 s).



Rinse. Remove excess water.
Do not dessicate tooth substance.



Apply LuxaBond. Remove excess material. Air blow gently.



Apply LuxaCore Z-Dual into the canal lumen.



Insert the LuxaPost.



Light cure for 20 s.



Build-up core with LuxaCore Z-Dual and light cure.

