
Icon Proximal

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






Lesion depth classification*

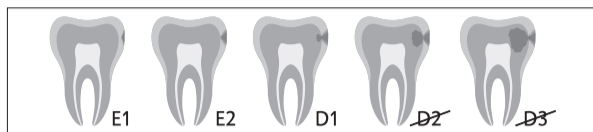* Radiographic lesion depth classification according to bitewing images

Handling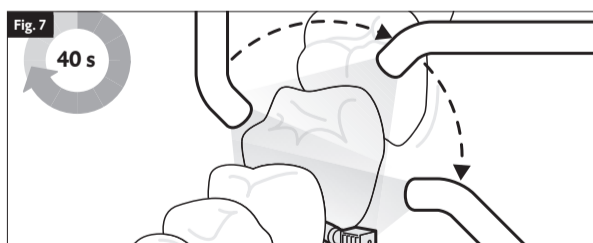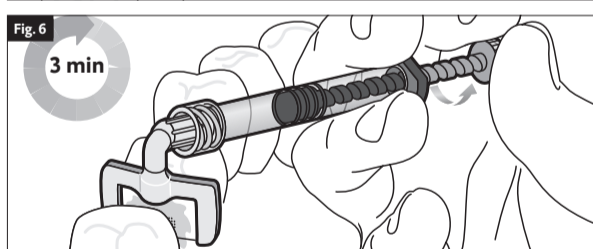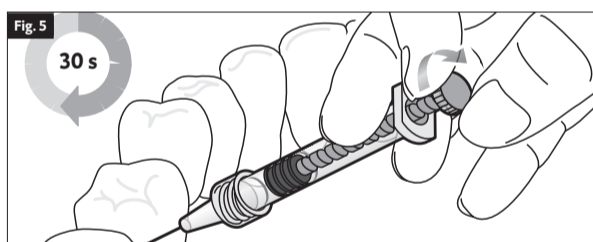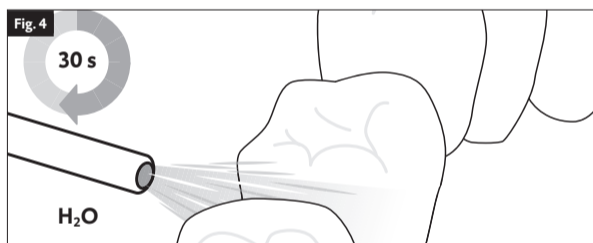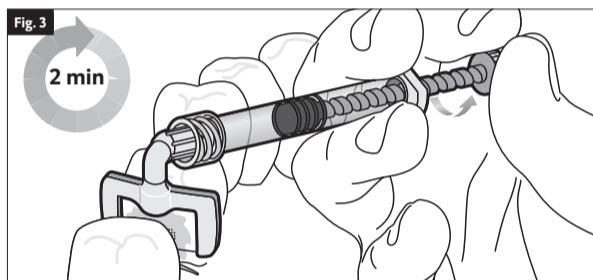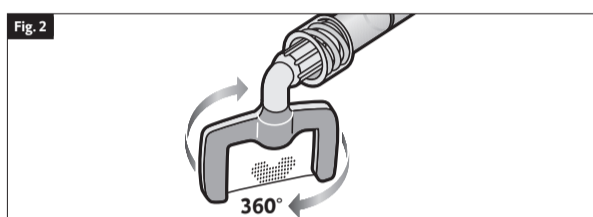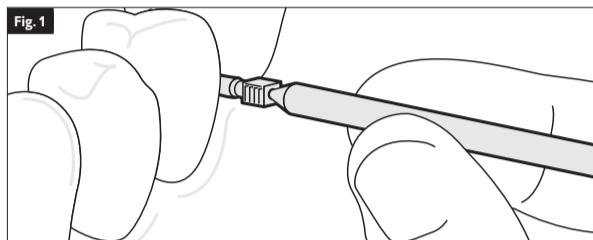

Patient card

Zahn Tooth	Fläche Surface	Lesionstiefe Lesion depth	3	E1	E2	D1	D2	D3	P	Behandlungsdatum Treatment date
			4	E1	E2	D1	D2	D3	P	Kontrolldatum 1 Follow-up appointment 1
			4	E1	E2	D1	D2	D3	P	Kontrolldatum 2 Follow-up appointment 2
			4	E1	E2	D1	D2	D3	P	
			4	E1	E2	D1	D2	D3	P	

Product description

Icon Proximal is an innovative product for the micro-invasive treatment of lesions in the proximal area.

Icon Proximal has been specially developed for the gentle treatment of incipient proximal caries.

Indications for use (Fig. see »Lesion depth classification«)

- Micro-invasive treatment of non-cavitated cariogenic lesions (lesion depth up to D1*)

* Radiographic lesion depth classification according to bitewing images

Contraindications

- Do not apply the material for deeper seated lesions (D2 – D3) (see »Lesion depth classification«) with cavitated enamel (enamel defects).
- Do not apply the material where there are known allergies or contact allergies to any of the components.

Limitations of use

The use of Icon is contraindicated if sufficient drying and protection of the gingiva is not possible.

Patient target group

Persons receiving treatment as part of a dental procedure.

Intended users

Dentist

Notes for use

- Light-curing units should emit at 450 nm and should be checked regularly.
- The light intensity should be a minimum of 800 mW/cm².
- Place the light unit as close to the material as possible.
- The etched surface must not be touched or contaminated with saliva or moisture before the treatment resumes. If contamination occurs after drying, re-etch for approx. 10 seconds and repeat drying with Icon Dry.
- For effective treatment results a sufficiently dry working area is critical. Therefore, appropriate measures for complete isolation must be taken. Do not use rubber dams made from thermoplastic elastomers such as Flexi-Dam (Coltène Whaledent/Hygenic).
- Discard syringes after use.
- The syringes included in the treatment unit contain sufficient material for treating two proximal lesions of one patient.
- Additional proximal surfaces can be treated consecutively during one visit.

Recommended use (for image, see »Handling«)

Preparation

1. Before the start of treatment, clean the affected tooth and adjacent teeth. Remove all cleaning residue with water spray.
2. Ensure that the working area is sufficiently dry. For this purpose, appropriate measures such as a rubber dam should be taken and used according to the manufacturer's instructions.

Separation

1. Introduce one of the enclosed dental wedges into the interproximal space (Fig. 1).
 - ▶ For better access to the proximal region the wedge handle can be bent or detached with a twisting motion.
 - ▶ In order to obtain sufficient separation of the teeth the wedges must be seated at an adequate depth in the interproximal space.
 - ▶ A separation of approx. 50 µm is recommended to ensure a successful treatment. This can be performed gradually: Insert the wedge to the point of resistance. Wait 3 to 5 seconds and gradually move the wedge deeper until sufficient separation is achieved.
 - ▶ Leave the wedge in the proximal space during the entire treatment procedure.
 - ▶ For tighter proximal areas, the use of an orthodontic rubber ring or other separation method prior to treatment (up to several days prior) will ensure adequate separation.

Exposing the lesion body

For effective treatment results the hypermineralization must be removed.

1. Screw the Proximal Tip onto the Icon Smooth Surface Etch syringe, and introduce the application tip into the proximal area. Make sure that the green side of the Proximal Tip faces in the direction of the surface to be treated (Fig. 2).
 - ▶ The material only dispenses on the green side of the Proximal Tip.
2. Apply Icon Smooth Surface Etch with a slight excess to the lesion site. 1.5 to 2 turns of the syringe shaft correspond approximately to the amount to be applied. Allow Icon Smooth Surface Etch to act for 02:00 minutes. Move the applicator slightly so that the etching gel will be activated. Remove excess material (Fig. 3).
3. Remove the application tip from the interproximal space. Aspirate off Icon Smooth Surface Etch and rinse with water for at least 30 seconds. Then dry thoroughly with air that is free of oil and water (Fig. 4).

Dry

For effective treatment results it is necessary to dry the lesion to its full depth.

1. Screw the application tip onto the Icon Dry syringe, apply an ample amount of material onto the lesion and allow to take effect for 30 seconds (Fig. 5). Dry carefully with oil-free and water-free air.

Infiltration

The lesion can be penetrated to its full depth by the Infiltrant after complete drying.

- ▶ Do not apply Icon Infiltrant under direct operating light as this may cause the material to set prematurely.
1. Screw a new Proximal Tip onto the Icon Infiltrant syringe, and introduce the application tip into the proximal area. Make sure that the green side of the Proximal Tip faces in the direction of the surface to be treated.
 - ▶ The material only dispenses on the green side of the Proximal Tip.
 2. Apply Icon Infiltrant with a slight excess to the lesion site. 1.5 to 2 turns of the syringe shaft correspond approximately to the amount to be applied.
 3. Allow Icon Infiltrant to penetrate for 03:00 minutes, occasionally activating the Infiltrant by gently moving the applicator and then top up as necessary (Fig. 6).
 4. Remove the application tip from the interproximal space. Remove excess material with dental floss.
 5. Light-cure Icon Infiltrant from all sides for at least 40 seconds.
 6. Screw a new Proximal Tip onto the Icon Infiltrant syringe, repeat the application, and let set for 01:00 minute. Remove the application tip from the interproximal space. Remove excess material with dental floss. Then light-cure from all sides for at least 40 seconds (Fig. 7).
 7. If a second lesion needs to be treated, repeat the recommended application described above for the affected site.
 8. Remove the wedge and rubber dam. Use polishing strips for the surface finish.

Interactions

Do not use together with substances which inhibit polymerization. This is especially valid for materials which contain eugenol, such as temporary cements.

Residual risks / side effects

- In some cases, contact allergies with similar composite products have been reported.
- Contact of Icon Smooth Surface Etch with the oral mucosa causes a white surface coloration. This will subside after a few days.

Warnings / precautions

- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Caution! Icon Smooth Surface Etch can cause chemical burns if not used properly.

- Avoid contact with soft oral tissue, eyes, and skin. In case of inadvertent contact rinse with copious amounts of water, and, if necessary, consult a physician.
- Limit the application to the surface to be treated. Avoid contact of the etching gel with gingiva, exposed dentin, root element and adjacent teeth.
- Use only the enclosed syringe and application tip to apply the etching gel in the patient's mouth.

- Discard the applicator tip after use. Replace syringe cap.

Performance characteristics of the device

Penetration coefficient: approx. 147 [cm/s]

Storage / disposal

- Store at 2-25 °C (36–77 °F) in a dry place.
- Protect against exposure to direct sunlight!
- Do not use after the expiration date!

Patient card + sticker (Fig. see »Patient card«)

This material is not radiopaque. For the documentation of the infiltrated surfaces and lesion depths, the enclosed patient card can be filled out at the time of the treatment and follow-up visits and be given to the patient. In addition, the enclosed sticker facilitates the documentation in the patient's record.

1. Front side: Enter the name of the patient and add the office stamp.
2. Tooth diagram: Mark the treated area.
3. Record the treated tooth [1], the treated surface [2] (d = distal, m = mesial, v = vestibular (buccal, cervical, labial), l = lingual or palatal), lesion depth*, and the treatment date [3].
4. Follow-up visit [4]: Record the lesion depth* and date of the follow-up appointment. Annual check-ups are recommended.

*see »Lesion depth classification«

Composition

- Icon Smooth Surface Etch: Hydrochloric acid (approx. 47 %), water (approx. 33 %), glycerol (approx. 15 %), pyrogenic silicic acid, surface-active substances
- Icon Dry: Ethanol (approx. 99 %)
- Icon Infiltrant: Methacrylate-based resin matrix (approx. 99%), initiators, additives

Note: A glossary of all symbols is displayed at: <http://www.dmg-america.com/literature>

Caution: US Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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