KOLORZ PROPHYLAXIS PASTE with Fluoride

INSTRUCTIONS FOR USE

DESCRIPTION: **KOLORZ Prophylaxis Paste** contains a unique blend of polishing and cleaning agents. It is for professional use only and is designed for application during standard dental practice hygiene procedures. Contains a concentration of 1.23% fluoride ion. These instructions are for all grits and flavors of **KOLORZ Prophylaxis Paste**. **KOLORZ Prophylaxis Paste** is Gluten Free!

INDICATIONS/RECOMMENDED USE: For cleaning and polishing procedures as part of a professionally administered prophylaxis treatment. NOTE: Prophylaxis pastes containing fluoride are not considered a replacement for other forms of fluoride therapy.

INSTRUCTIONS TO APPLY KOLORZ PROPHYLAXIS PASTE:

Each individual cup contains enough **KOLORZ Propphy Paste** for a single prophylaxis procedure. Use any commercially available prophy angle, rubber cup combination.

1. Fill the cup by pressing the face of the cup into the paste while running the angle at low speed.
2. Throughout the procedure, enough paste should be maintained on the cup to avoid overheating of the tooth surface.
3. Saliva evacuation is recommended to avoid excessive dilution and swallowing of the paste.

PRECAUTIONS, WARNINGS AND RISKS: For use by a medical or dental professional. For use in the oral cavity. Excessive ingestion of fluoride can cause nausea. **KOLORZ Prophylaxis Paste** should be expectorated and not swallowed.

CONTRAINDICATIONS: **KOLORZ Prophylaxis Paste** is contraindicated in patients with a known allergy to any of the components.

MEDICINAL INGREDIENTS (per dose unit): Each individual cup contains 2.72% Sodium Fluoride (1.23% Fluoride Ion).

NON-MEDICINAL INGREDIENTS: Glycerin, Phosphoric Acid, Potassium Sorbate, Pumice, Purified water, Sodium silicate, Sucralose, Xylitol, Agar, Flavors, Cinnamon: FD&C Yellow # 6, FD&C Red # 40, Bubblegum: FD&C Red # 28, Mint FD&C Blue # 1, FD&C Yellow# 5, Cotton Candy: FD&C Red # 40, Blue Raspberry: FD&C Blue # 1, Cherry Burst: FD&C Red # 40.

| | |
|---------------------------|---|
| Distributed by: | DMG America, LLC 242 S. Dean St., Englewood NJ 07631 Tél : 800 662-6383 |
| Canadian Importer: | Progress Therapeutics Inc. 320 Harry Walker Pkwy North, Suite #14 Newmarket ON L3Y 7B4 |
| License Holder: | Dental Savings Club 31, rue Serge-Garant Verdun (Québec) H3E 0A6 Canada |

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Fabriqué pour : **DMG America, LLC**
Importateur canadien : **Progress Therapeutics Inc.**