TempoCemNE

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English

Product description

TempoCemNE is a eugenol-free automix cement based on zinc oxide that is indicated for temporary cementation crowns and bridges, inlays and onlays, as well as all types of temporary restorations.

Indications

TempoCemNE

- Temporary cementation of provisional and permanent crowns and bridges, inlays and onlays.

Contra-indications

Do not use the material in cases of allergies to any of the substances contained or, as can occur in rare cases, in the event of contact allergy.

Basic safety instructions

- Only for dental use!
- Keep out of reach of children!
- Avoid contact with skin and eyes! In the event of accidental contact, rinse thoroughly with plenty of water immediately and consult a physician if necessary.
- Contains rosin

Application System

- Automix cartridge: see "Using the Automix cartridge"
- Smartmix syringe: see "Using the Smartmix syringe"

Please note: When using a cartridge or syringe for the first time, squeeze out a pea-sized amount of material and discard it.

Timing

TempoCemNE

≈1:00 min	Working time
4:00 - 5:00 min	Curing time in the mouth


Please note: The times given apply at a room temperature of 23 °C and a normal relative air humidity of 50 %. Higher temperatures shorten, lower temperatures increase these times.

Recommended Application

Temporary cementation

1. Apply TempoCemNE to the dry temporary restoration.
2. Insert the temporary restoration within 1:00 min of start of mixing and attach to the dry, prepared tooth using slight pressure.
3. After curing, remove excess cement with a probe.
4. It is easy to remove excess cement in large chunks.
5. Remove interdental excess by flossing carefully.
6. The light color of the material makes it easy to distinguish from teeth.

Using the Automix cartridge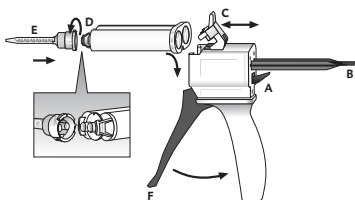

Insert cartridge

1. Push up the lever [A] at the back of the Automix dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C], insert the cartridge and lock it using the plastic lever [C].

Please note: Make sure that the notches on the cartridge line up with those on the Automix dispenser.

Attach mixing tip

1. Remove the cartridge cap [D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

Please note: Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.

Apply material


- ▶ When using a cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- ▶ The material is mixed in the mixing tip by pulling the trigger [F] of the Automix dispenser and it is then applied directly.

Please note: After the application, leave the used mixing tip on the cartridge to cap unused material!


Remove cartridge

1. Push up the lever [A] at the back of the Automix dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C] and remove the cartridge.

Using the Smartmix syringe

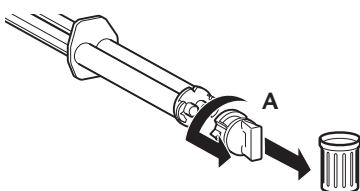
1. Before putting on the mixing cannula, remove the cap [A] or used mixing tip after twisting it 90° counterclockwise, and then discard it.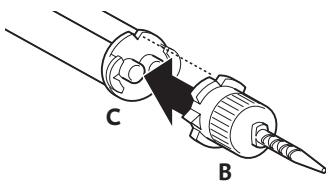

2. Attach a new mixing tip [B].

Please note: Make sure that the notches on the mixing tip line up with those on the Smartmix syringe [C].

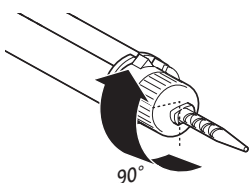3. Turn the mixing tip clockwise 90° to lock it.
4. The material is mixed in the cannula when it is expressed and it can be applied directly.

Please note: When using an syringe for the first time, squeeze out a pea-sized amount of material and discard it!

Leave the used mixing cannula on the Smartmix syringe after use. It serves as a cap!

Disinfection of the Automix dispenser

The Automix dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute). The Automix dispenser is also autoclavable.

Side effects

Isolated cases of hypersensitivity or contact allergy to components of the material cannot be excluded.

Composition

- Catalyst paste: Natural resins, fatty acids, additives
- Base paste: Zinc oxide, paraffin, additives
- Available in 5 ml and 25 ml

Classification

ISO 3107:2011, type 1 (because of the automatic mixing, the curing time is shorter than usual)

Storage and shelf life

- ▶ Store at room temperature (15 to 25 °C/59 to 77 °F) in a dry place!
- ▶ Do not store in the refrigerator!

- ▶ Protect against exposure to direct sunlight!
- ▶ After the application, leave the used mixing tip on the Automix cartridge to cap unused material!
- ▶ Leave the used mixing cannula on the Smartmix syringe after use. It serves as a cap!
- ▶ Do not use after the expiration date!

Note: A glossary of all symbols displayed is at: <http://www.dmg-america.com/literature>

Caution: U.S. Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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