

TEMP TABS®-*TRUE BLUE*

TEMP TABS®-*NATURAL*

BITE BUDDY

INSTRUCTIONS:

1. Heat one **TEMP TABS®-*TRUE BLUE*** in 150°F water for a single unit. Two Tabs may be joined in the transparent state and then shaped to fit multiple units (up to 5).
2. After the Tab has become transparent (about 1 minute), remove from water. **Keep fingers moist** to place softened Tab on the occlusal surface. Set up is quick. No pre-shaping is necessary! Press down and roll excess into the margins.
3. Let the Tab cool until it regains its opaque and rigid state (20 seconds with an air syringe).
4. Remove the cooled Tab from the mouth.
5. Inject temporary Crown & Bridge material according to the manufacturer's instructions.

IMPORTANT NOTES:

- Use a **separating medium** with C&B materials containing MMA.
- **TEMP TABS®-*TRUE BLUE*** is suitable for impressions of up to 5 units.
- Remove temporary from matrix at its elastic phase. **Do not let any temporary material go to full set inside the matrix.**